Instructions for use Calma Breastmilk bottle with teat

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Dear parents, Congratulations on your decision to purchase Calma. For almost 50 years, Medela has pioneered innovative products which help to enhance babies' health through the life-giving benefits of breastmilk. With Calma you can trust in a product that has been developed on a well-established research foundation. This research has shown that your baby is able to maintain with Calma the same learnt feeding behaviour that your baby exhibits when breastfeeding. Your baby will always prefer and benefit from breastfeeding. However, if you are separated from your baby, Calma provides an excellent alternative that stands to minimize any interference with the breastfeeding relationship between you and your baby.

Yes, it is the right length! Even if you think that Calma looks a bit too long, don't worry, it is the perfect length. As you know, your nipple lengthens up to 2 times in your baby's mouth while breastfeeding. As such, Calma reaches a similar spot in your baby's mouth as the breast. It is, however, important that your baby has the right latch-on position (see picture 1).

Do you know that your baby has to do some work? During breastfeeding, milk does not flow automatically. To facilitate milk flow, your baby has to create a vacuum. During breastfeeding milk flows when the vacuum in your baby's mouth is at its greatest level. At this time the tongue of your baby is at the lowest point and your nipple is drawn into your baby's mouth. This is the same with Calma. Your baby has to work to build up a vacuum for milk to flow (see picture 2).

First time right You might not believe it, but one size is sufficient. Just as your breasts are the same over the course of your lactation. The flow, shape and length of Calma are designed to suit your baby's needs as your baby grows. Calma will always meet your baby's needs during the lactation period.

Your baby has time to breathe Your baby has a unique sucking pattern, with pauses to swallow and breathe. By using Calma, your baby is able to feed and pause naturally according to his personal rhythm. This helps to slow down the feed and results in a calm and relaxed baby during and after the feed.

Can Calma help avoid gassiness? Calma helps avoid gassiness as any other vented teat bottles as the bottle is vented through our unique air control system (see picture 3). However unlike many other vented bottles Calma vents air independent of how it is assembled.

Calma does not drip! You can turn it upside down and it still won't drip. Your baby needs to apply a certain vacuum in order to get milk out. It is, however, possible that there is some dripping of milk out of the teat at the end of the feed if baby gets off the teat before drinking the portion of milk just drawn from the bottle.

Where does the milk come from? The milk flows through the small hole (see picture 4) at a rate that is controlled by the interaction of your baby's vacuum and the milk-flow control system. Compressing the teat will not result in any milk flow. You can be confident your baby will be able to get enough milk.

Why is the hole at the tip of the silicone so wide? Only the milk that passes through the milk-flow control system will flow into the baby's mouth. The wide hole makes the teat soft, flexible and adaptable to your baby (see picture 5). Hence your baby can drink with his own feeding action.

When feeding your baby with Calma, try to use a different position than the one you use with breastfeeding (e.g. with your baby facing the carer). This may help your baby to accept Calma and make it easier for you so that your baby will come back to the breast.

Your baby will always prefer and benefit from breastfeeding so breastfeed your baby whenever you are together. For those times you are separated from your baby we do recommend you to use Calma exclusively and not in conjunction with other teats when feeding your baby from a bottle. If your baby has already used another teat, be aware, that with Calma, the milk flow is controlled by the interaction of your baby's sucking action and the milk-flow control system. This is different than with any other teat - you may need a bit of patience.

Medela wishes you and your baby all the best!

Your feedback is important for us. We would be very pleased if you could give us your response on www.medela.com/calma. Here you can take part in the draw and win a prize.

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Beginnen Sie Ihr Kind erst mit einem Sauger zu füttern, wenn das Stillen gut etabliert ist.
Intended use
Breastfeeding babies are able to feed calmly and breathe regularly and maintain high blood oxygen saturation levels. Babies feeding with Calma are also able to feed calmly and breathe regularly and maintain these same high blood oxygen saturation levels.

Important Safeguards
Read all instructions before use
Follow these cleaning instructions.
Inspect before each use and pull the teat and different parts in all directions. Throw away at the first signs of damage or weakness.
Always use this product under adult supervision. All parts which are not in use must be kept out of children’s reach.
Never use a teat as a soother.
Continuous and prolonged sucking of fluids can cause tooth decay.
Always check the temperature of food before feeding it to your baby (risk of burning).
Do not warm milk in a microwave oven as this may cause uneven heating and could scald your baby.
SAVE THIS INSTRUCTION MANUAL

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